

# 8D

# HIFU PRO






# Work Theory



**8D HIFU PRO is Non Surgical Face Lift Treatment- 8D HIFU PRO (High Intensity Focused Ultrasound) Technology**, the aging effects will show on the skin like the wrinkles and the eye-bags. It is the signals to alert you your collagen and elastin are losing from your body. The collagen is the key structural component to support your skin.

**8D HIFU PRO** (High-Intensity Focused Ultrasound) which is the new technical for face lifting and tightening.

8D HIFU Pro is delivered into the deep skin and cause thermal coagulation to SMAS (Superficial Muscular Aponeurotic System). It can provide the skin tightening and rejuvenation happens gradually with the time.

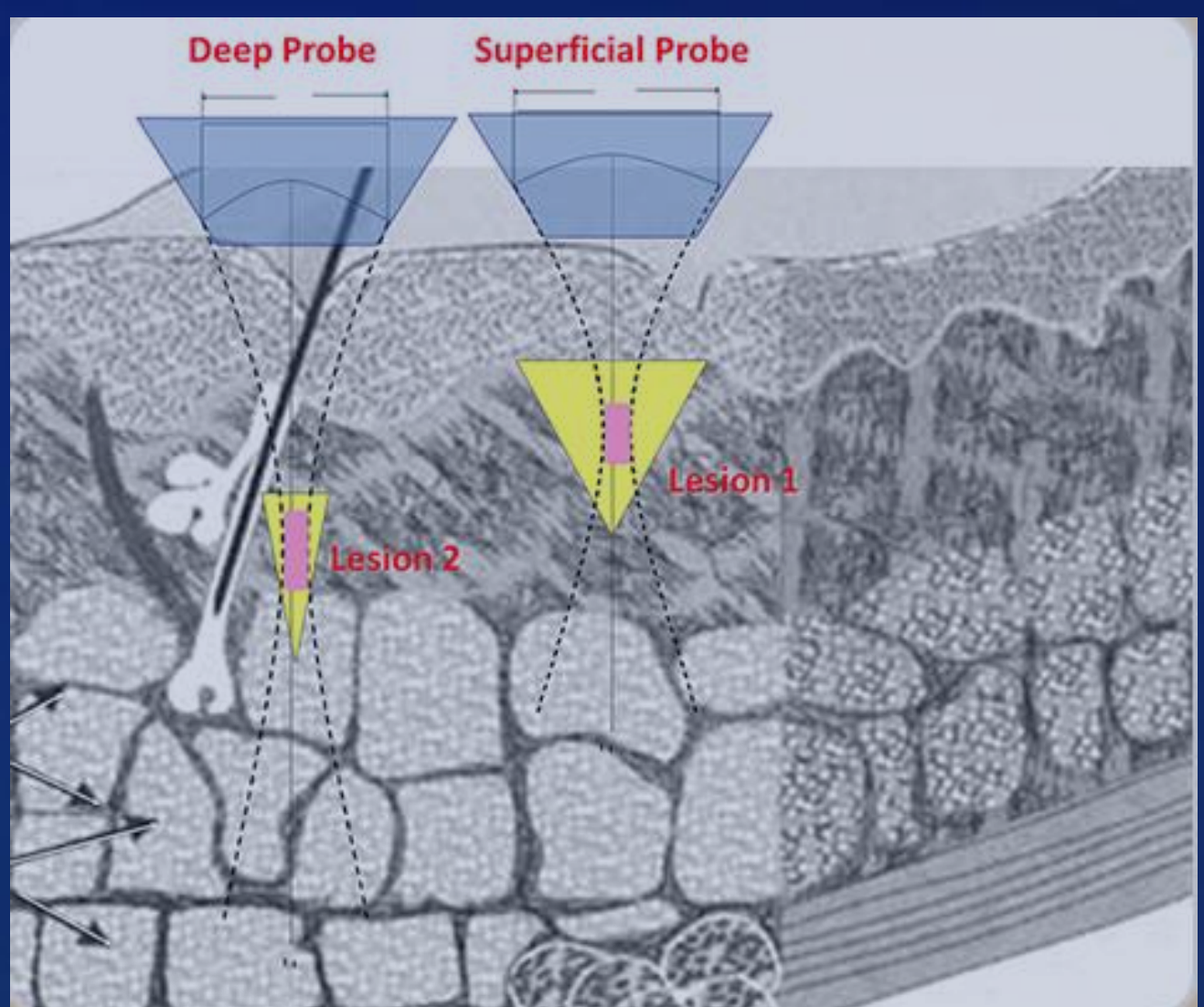

8D HIFU PRO technology device gives effective, safe and painless skin tightening.



# Effect



**Strengthen and tighten  
relaxed skin with  
remarkable tightening  
effect**

01

**Only one treatment is  
needed to smooth  
wrinkles and improve  
them comprehensively.**

02

**Make skin texture more  
delicate and glossy**

03

**Filling in shrinkage and  
loss of collagen  
vacancies to enhance the  
relaxation profile**

04

**Remove the traces of age  
such as roughness, and  
Skin rejuvenation**

05

**Make skin tender,  
white and youthful**

06





# Specifications

Frequency	Power
4MHz (Single line 1.5mm/2.0mm:7MHz)	350W
Treatment Depth	Treatment Length
S1.5mm S3.0mm S4.5mm WD 3.0mm WD4.5mm WD6.0mm WD13.0mm R1.5mm R3.0mm R4.5mm	5-25mm
Width	Energy
0--12mm	0.1-- 3.0J
Spacing	Power Source
1.0mm-5.0mm	AC 110V~220V--50Hz/60Hz



# System introduction



# Advantages



Completely non-invasive and non-ablative to the epidermis layer



No down time, and make-up can be wearied only in 15 min after the treatment



Fast operation time , full face and neck only takes 40-60 minutes



Long lasting result, 2-3 year can last only with one treatment



Flexible treatment, one single treatment achieves instant result. High quality, low consumables









# FL HP



**Treatment:** Body

Multi Line points: 20000



# FLHP

## Multi Line Options



3 mm



4.5 mm



5.5 mm



6 mm



9 mm



12 mm



13 mm



# FL HP



**Treatment:** Face, Neck, Jaw

Single Line points: 20000



# FL HP

## Single Line Options



1.5mm



2.0 mm



3.0mm



4.5 mm



# SD HP



**Treatment:** Around eyes and mouth

Single Dot Points: 40000



# SD HP



1.5 mm



3.0 mm



4.5 mm



# Q&A



**Who is suitable for 8D  
HIFU PRO treatment ?**

It suit for 30 to 60 years old customer  
who want to reduce the pore size and  
looks younger.



**when you will see the  
treatment result?**

The immediate effect is 10%-30%,  
and the best effect is in 3-6 months



# Q&A

**why some clients have good effect of face lifting, but some clients have hollow cheek after treatment?**

80% effect comes from the after treatment recovery, it's depend on patients skin property and recruitment of cell-repairing factor (such as DPHP)

( If patients age more than 30 years old, the collagen will loss more and more seriously, so before treatment, they need drink some Special repair of collagen ACME TEA so that can help regeneration of collagen fibers)

During these time, If Patients didn't support enough collagen timely, due to the deficiency of regeneration collagen will result in hollow cheek.



# Q&A



**If a patient undergo 8D HIFU PRO treatment too frequently, what risk he will take ? what is the suitable interval time for 8D HIFU PRO treatment?**

Usually the treatment results of 8D HIFU PRO treatment can last 2-3 years. 8D HIFU PRO treatment has a little damage on Subcutaneous, so we don't suggest a patient accept 8D HIFU PRO treatment too frequently, we suggest the interval time is above 2 years.



**How to do the postoperative care after 8D HIFU PRO treatment?**

After the treatment, the patient can do the daily life. There is no special considerations. The treatment area maybe swelling, but don't worry, it will disappear after a few hours.







# UI Interface



Single Line

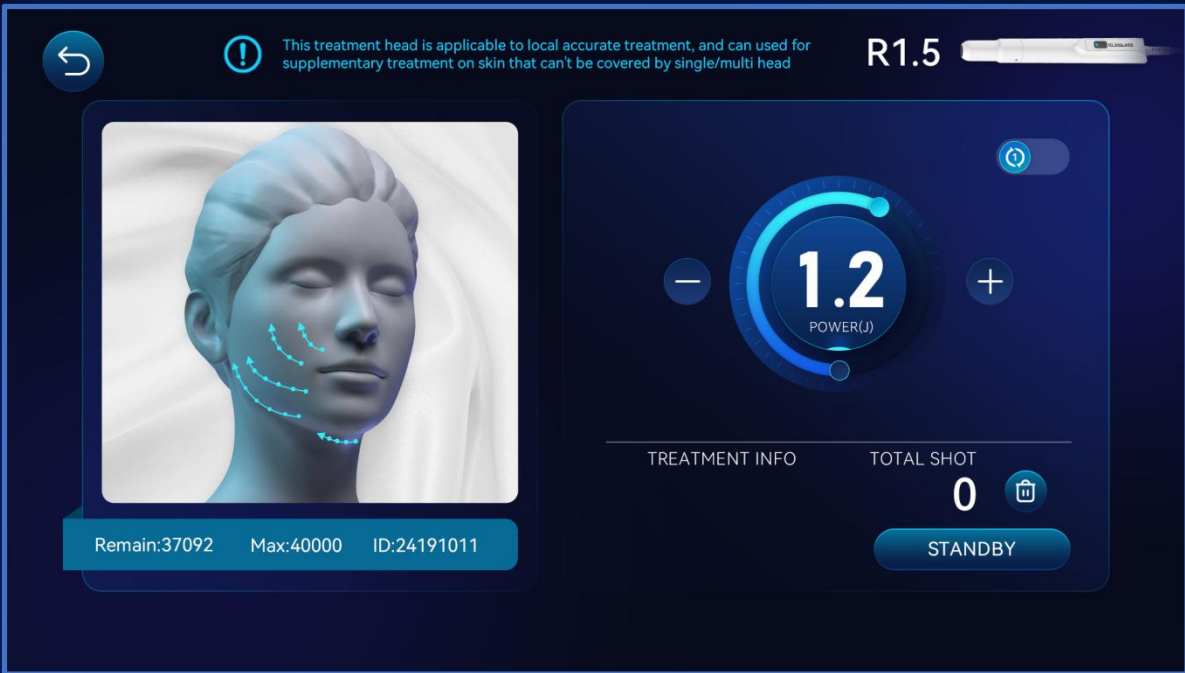

Single Dot

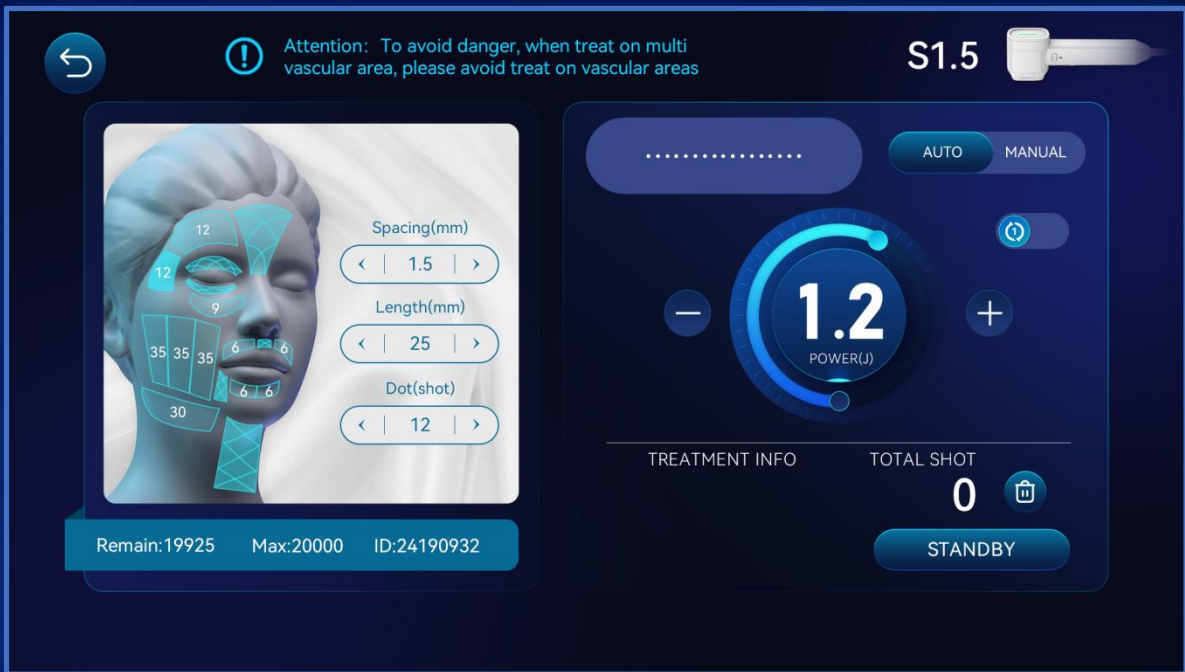

Multi Line



Multi Line






# Before & After



Before / After




Before / After

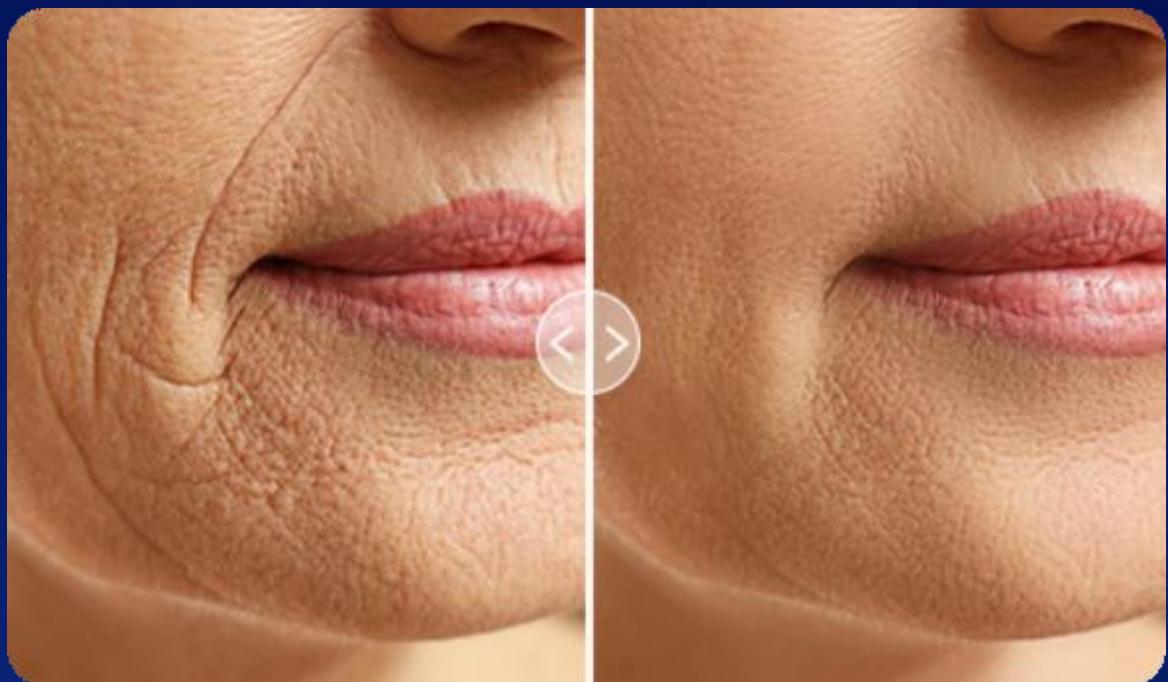

Before / After



Before / After

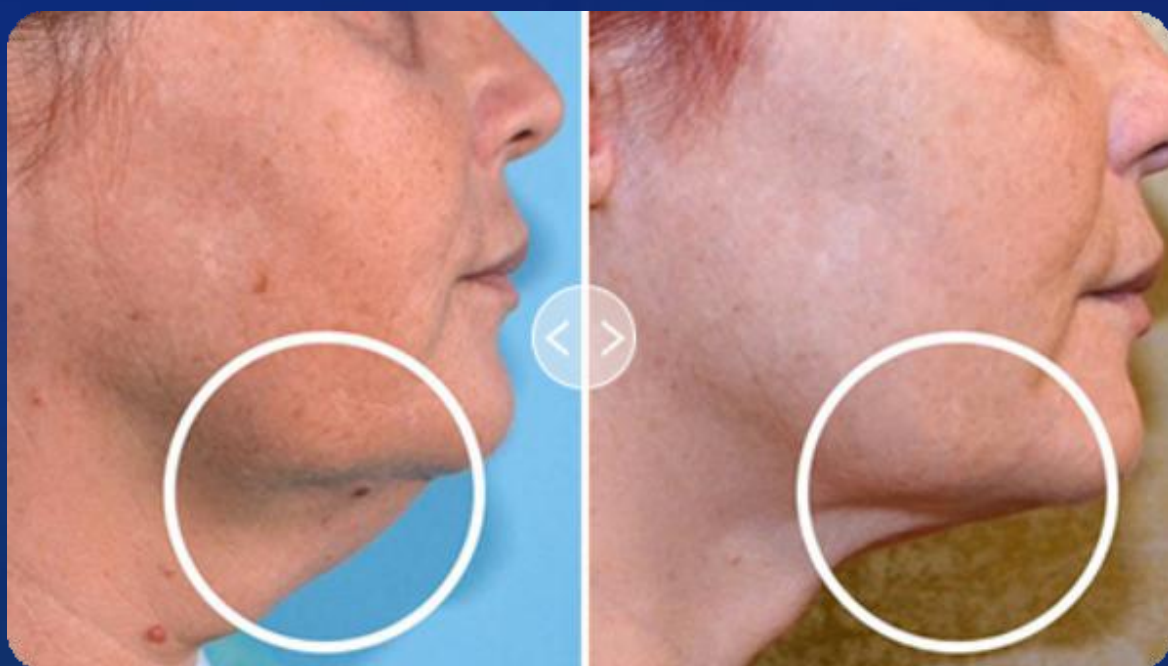

Before / After



Before / After