ASLD

Technical specifications

Intended use	Endodontics						
Indications for use	Endodontic apical surgery						
Description	Diamond-coated left-angled tip for the retro-treatment of pre-molars and molars						
Tip type	Diamond-coated						
Allergenic factor	Tip made of surgical stainless steel as per ASTM F899 - 12b Surgical stainless steel contains: • nickel • chromium Diamond polishing crimping binder alloy contains: • nickel						
Commercial reference	ASLD : F00080						
Medical class	Class of medical device: IIa according to 93/42/EEC directive						
Adjustments	Newtron range	P5 Range	Suprasson P5 Booster		y Max - P-Max - nter 2 - Piezotome 2	Irrigation	
	7	7	5	3		Ń	

Service life

As it is not possible to establish a maximum number of uses (that may be determined by many parameters such as duration of use, hardness of tissue, the force applied, wear), we recommend that medical devices being routinely used are replaced at least once a year. Replace the medical device if the tip does not vibrate at the expected frequency, the treatment is not progressing as normal and is taking longer or is at a standstill.

Associated documentation

Document title	References
Consulting electronic user instructions	J00007
Cleaning, disinfection and sterilisation instructions for tips	J02001
Endosuccess Apical Surgery Kit clinical tips	J08161
Ultrasonic generator power settings table	J58000
General instructions relating to the complete range of standard ultrasonic dental tips	J02101


This document is an English translation of the original French version. Reference J00470 version V3 and drawing number NB15FR050C




ASLD

Sterilisation

Sterilise prior to each use

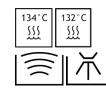

The tips must be cleaned, disinfected and sterilised prior to each use. Prior to each use, please refer to the cleaning, disinfection and sterilisation instructions for tips J02001.

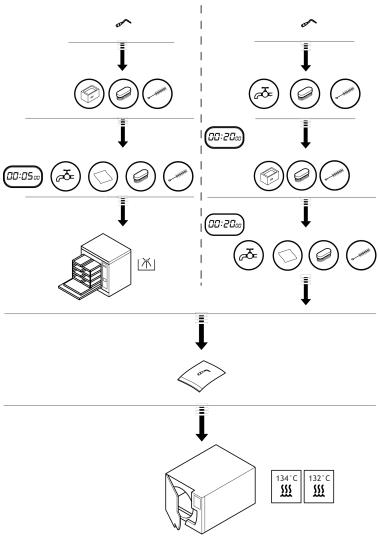




The practitioner has to continually monitor the clinical site to immediately identify any heating risk.

Tip | ASLD | J00471 | (09) | V3 | 05/2021 | NB15EN050C

SATELEC A Company of ACTEON Group 17 av. Gustave Eiffel | ZI du Phare 33700 MERIGNAC FRANCE Tel. +33 (0) 556 34 06 07 | Fax. +33 (0) 556 34 92 92 satelec@acteongroup.com|www.acteongroup.com

Installation

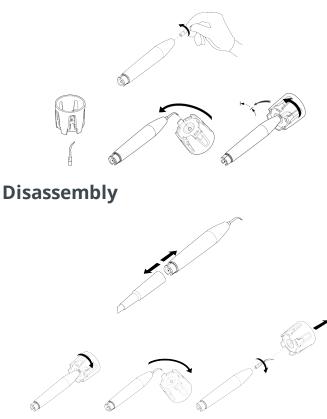

Manufacturer identification



SATELEC

A Company of ACTEON Group 17, avenue Gustave Eiffel ZI du Phare

33700 MERIGNAC France Tel. +33 (0) 556.34.06.07 Fax. +33 (0) 556.34.92.92 E.mail:satelec@acteongroup.com www.acteongroup.com



Date of first CE marking

2009



